




Digital Implant Library

Step-by-Step Integration & Product System Selection Manual

This technical manual provides complete instructions for installing, configuring, and mapping the **Kamal Dental Implant Libraries** within the **exocad CAD/CAM** environment (including DentalCAD and Model Creator modules).

1. DIRECTORY STRUCTURE & FOLDER HIERARCHY

The Kamal Digital Implant Library installation package is structured to support both **Implant Design** and **Model Creator** modules in exocad:

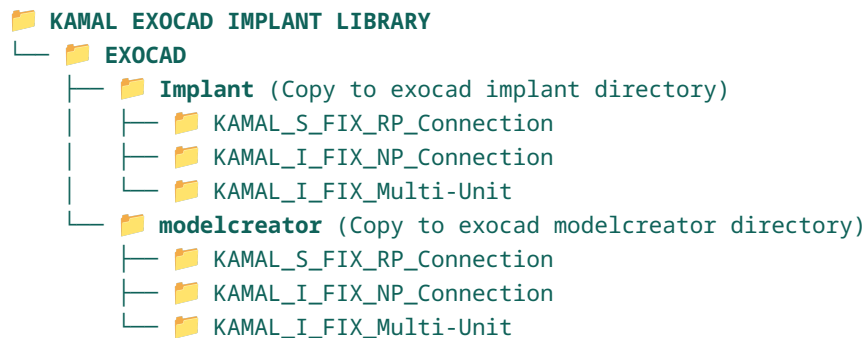

2. PRODUCT PLATFORM & LIBRARY SELECTION MATRIX

Select the appropriate library folder in exocad matching your specific implant line and connection platform:

Platform System	Covered Products & Series	exocad Library Folder Name
RP CONNECTION Regular Platform	• S-Fix (S-Series) • S-Fix Pro (SP-Series)	KAMAL_S_FIX_RP_Connection
NP CONNECTION Narrow Platform	• I-Fix (I-Series) • I-Fix Pro (IP-Series) • AK Series (Apex Konnect)	KAMAL_I_FIX_NP_Connection
MU CONNECTION Multi-Unit Platform	• Common to all Platforms (RP & NP Multi-Unit Restorations)	KAMAL_I_FIX_Multi-Unit

3. INSTALLATION WORKFLOW STEPS

1 Locate exocad Application Library Path:

Open Windows File Explorer and navigate to your exocad installation directory.

```
Standard Path: C:\DentalCADDB\DentalCADApp\library\
```

2 Install CAD Implant Libraries:

Copy the 3 folders inside .../EXOCAD/Implant/ and paste them into:

```
Target Path: C:\DentalCADDB\DentalCADApp\library\implant\
```

3 Install Model Creator Libraries:

Copy the 3 folders inside .../EXOCAD/modelcreator/ and paste them into:

```
Target Path: C:\DentalCADDB\DentalCADApp\library\modelcreator\implant\
```

4 Restart & Verify:

Restart **DentalDB** and **exocad DentalCAD**. Select the Kamal libraries from the drop-down menu during implant setup.

Pro Tip / Note:

Ensure exocad is closed before pasting folders. Grant Administrator permissions if prompted by Windows.

TECHNICAL SUPPORT & ENQUIRIES

If you require assistance during installation or library setup, please contact:

Contact Person: Mr. Prashant Iyer

Phone Number: +91-8104649865



LEGAL NOTICE & INTELLECTUAL PROPERTY

All copyrights, registered trademarks, brand assets, product logos, CAD geometries, and digital library data contained within this distribution package belong exclusively to **Kamal i-Fix Dental Implants P/O Kamal Medtech**. All rights reserved.

Unauthorized copying, reproduction, re-engineering, or redistribution of these digital assets without prior written consent from **Kamal i-Fix Dental Implants P/O Kamal Medtech** is strictly prohibited.

Third-Party Trademark Acknowledgment:

exocad® is a registered trademark of exocad GmbH. All third-party names, marks, and brands referenced in this manual are used solely for identification, compatibility, and software integration purposes.